

General Instructions

- ◆ Ensure camera lens clean. Use Lenspen.
- ◆ Ensure patient is comfortable. Adjust table, chair and chin rest height as needed
- ◆ Ensure photography room is completely dark. Reduce brightness of monitors, close doors, and close window coverings.
- ◆ If the patient cannot see the fixation light press the Set button to make the internal fixation light blink.
- ◆ (For non-mydriatic photography) Wait sufficient time between photos for the pupil to recover naturally to achieve good quality photographs. Alternate between the right and left eye.

Photography Procedure

Right field 1: Move the stage to your left; Press FIX TARGET button to the left; Instruct patient to look to their left to center the optic disc; Capture

Left field 1: Move the stage to your right; Press FIX TARGET button to the right; Instruct patient to look to their right to center the optic disc; Capture

Right field 2: Move the stage to your left; Press FIX TARGET button to the right; Instruct patient to look forward to center the macula; Capture

Left field 2: Move the stage to your right; Press FIX TARGET button to the left; Instruct patient to look forward to center the macula; Capture

Quality Checklist

- Positioning
- Illumination
- Focus and clarity (tertiary vessels visible)
- Artifact-free (no dust or smudges)

